

What is Cryo Professional?

Cryo Professional is a professional cryosurgery product intended to be used for the treatment of benign skin lesions in patients over 4 years old.

How does Cryo Professional work?

Cryo Professional is a cryosurgery product indicated for the treatment of various types of benign skin lesions. Cryotherapy freezes the skin lesion whereby it necrotises. The product contains dimethylether

with propane / isobutane. Cryotherapy is a known effective and reliable treatment.

With Cryo Professional, different types of benign skin lesions can be treated i.e., warts on hands or feet, genital warts, skin tags, water warts, age spots and hyper pigmentation. The frequency and duration of the treatment varies per type of skin lesion and ranges from 15 to 40 seconds. Refer to the instruction for use for the specific treatment times.

Table of recommended treatment times per skin lesion

Type of skin lesion	Recommended freezing times
Genital warts	40 seconds
Molluscum Contagiosum	20 seconds
Verruca (Plantaris, Vulgaris) (warts on hands or feet)	40 seconds
Verruca (Plana)(flat warts)	20 seconds
Fibroma, Acrochordon (skin tags)	40 seconds
Seborrheic keratosis (SKs)	40 seconds
Aktinica keratosis (AKs) -> on the face	15 seconds
Aktinica keratosis (AKs) -> on other body parts	40 seconds
Lentigo -> on the face	15 seconds
Lentigo -> on other body parts	40 seconds

